



SARS-CoV-2 S-RBD Antibodies Test

(Colloidal Gold)

The SARS-CoV-2 S-RBD Antibodies Test (Colloidal Gold) is intended for the qualitative detection of the SARS-CoV-2 S-RBD Antibodies in human serum, plasma and whole blood sampled from individuals after vaccination or after the infection with SARS-CoV-2.

- ✓ **Simple**
- ✓ **Rapid**
- ✓ **Convenient**

This kit is intended for professional use by trained personnel.



PRODUCT FEATURES



Sensitivity




Specificity



Test time


INSTRUCTION

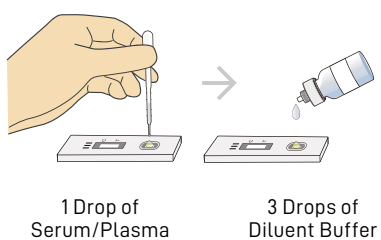

1 Drop of
Serum/Plasma

3 Drops of
Diluent Buffer

Result

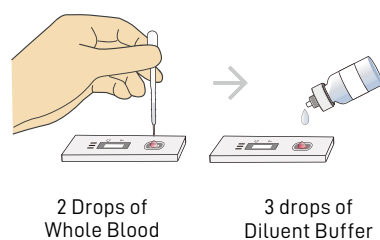


2 Drops of
Whole Blood


3 drops of
Diluent Buffer

Result


RESULT INTERPRETATION

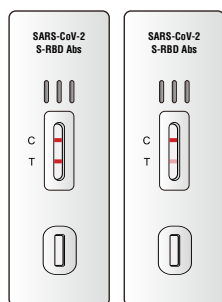

POSITIVE

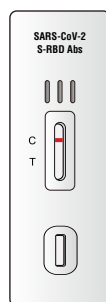

NEGATIVE

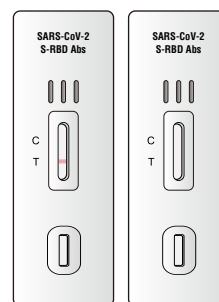

INVALID

ORDER INFORMATION

No.	Product Name	Spec.
S3132E-25	SARS-CoV-2 S-RBD Antibodies Test (Colloidal Gold)	25T/kit

The kit components: SARS-CoV-2 S-RBD Antibodies Test Cassette; Diluent Buffer; Blood Lancet for Single Use; Dropper; Alcohol Pad.

